

# TANITA

## Wireless Adapter OP-102-RU

### Setup Guide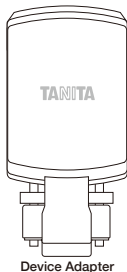

Device Adapter

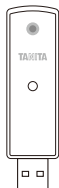PC Adapter

This is the instructions for the wireless PC and device adapters.

Please read this Setup Guide carefully and keep it for future reference.

TANITA is not responsible for damage or loss of data due to improper operations and resumes no responsibility for any resulting damages. Images in this guide are for illustration purposes only.

#### For Your Safety

This section explains precautionary measures to be taken to avoid injury to the users of this product and others, and to prevent damage to property. Please familiarize yourself with this information to ensure safe operation of this product.

<p> <b>Warning</b> Failure to follow instructions highlighted with this mark could result in death or severe injury.</p> <p> This mark indicates actions that are prohibited.</p>	<p> <b>Caution</b> Failure to follow instructions highlighted with this mark could result in injury or damage to property.</p> <p> This mark indicates instructions that must always be followed.</p>
---	---


#### Warning

<p> Prohibited</p> <ul style="list-style-type: none"> <li>• <b>Do not disassemble or perform modifications.</b> Doing so may lead to damage to the equipment, electric shock, or injury.</li> <li>• <b>Do not plug or unplug the power plug or other connectors when hands are wet.</b> Doing so may lead to electric shock, fire or short circuit.</li> <li>• <b>Do not daisy chain or otherwise overload electrical outlets.</b> Doing so may lead to fire.</li> <li>• <b>Do not subject the device to excessive impact or vibration.</b> Doing so may lead to malfunction.</li> <li>• <b>Do not put water, chemicals, oils or any other foreign matter inside the device.</b> Doing so may lead to fire, electric shock or malfunction.</li> <li>• <b>Do not leave in reach of infants or small children.</b> Small parts are a choking hazard.</li> <li>• <b>Do not use in areas where the use of wireless devices is prohibited.</b> Failure to heed this warning may lead to serious accident.</li> <li>• <b>Those using pacemakers and other electronic medical equipment should consult their physician before using this device.</b> Electronic medical devices may malfunction due to radio wave interference, which may lead to serious accident.</li> </ul>	<p> Required</p> <ul style="list-style-type: none"> <li>• <b>Periodically remove any dust from the power plug and other connectors.</b> Build-up of dust can lead to accumulation of moisture and other issues that cause insulation failure, which may lead fire.</li> <li>• <b>Do not use the AC adapter cable while it is wrapped up.</b> Doing so generates heat which may lead to fire.</li> <li>• <b>Should any of the following occur, unplug the connector and AC adapter, and refrain from use.</b> - Equipment is smoking.      - Equipment is generating abnormal noises or odors.</li> <li>• <b>Use this device with a dedicated TANITA equipment.</b></li> </ul>
--	--

## Part Names & Accessories


### Device Adapter

Front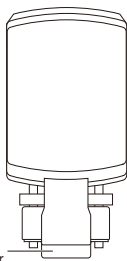

RS-232C  
Connector

Side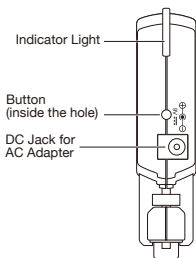


Indicator Light

Button  
(inside the hole)

DC Jack for  
AC Adapter

Back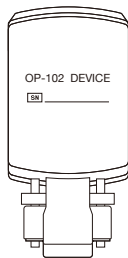


OP-102 DEVICE

SN

### PC Adapter

Front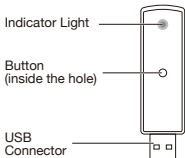


Indicator Light

Button  
(inside the hole)

USB  
Connector

Back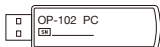


OP-102 PC

SN



✓ Setup Guide (this guide)

## Setting

RS-232C Port  
(TANITA equipment)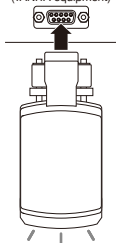

USB Port  
(PC)

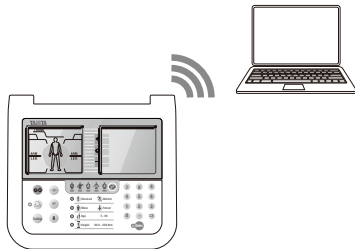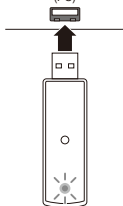1. Set up the **device adapter** on the body composition analyzer and set up the **PC adapter** on your computer. Connection starts once power is supplied. The lamps blink green when connection starts.

- When connection is completed, the adapter lamps remain lit in green.
- Measure body composition. The measurement results are sent to the PC when they are displayed on the body composition analyzer. The lamps remain lit in blue while measurement data is being sent.

#### Note

The adapters are paired as the factory default. You must install the necessary driver onto your PC, available download from <https://www.tanita.eu>. See the separate document for further information on changing the power supply settings.

#### Note

When communication is disconnected, the adapter lamps remain lit in red.

## Indicator Lights


The green light blinks quickly.	Pairing	The green light remains lit.	Connected
The green light blinks.	Connecting	The blue light remains lit.	Transmitting data

## Specifications


Model number	OP-102-RU	
	Device Adapter	PC Adapter
Power source	9V DC (center negative)	
Electric current range	500mA or less	
Product size	93×44×22mm	72×20×9mm
Usage conditions temperature	0 to 35°C	
Usage conditions relative humidity	30 to 80 % (without condensation)	
Communication specification	Communication standard	EIA RS-232C
	Communication system	Asynchronous
	Signal speed	9600 bps
	Data bit length	8 bit
	Parity	None
	Stop bit	1bit
	Flow control	None
Wireless device specification	Radio equipment type	Bluetooth®4.1
	Frequency band	2402 to 2480MHz
	Maximum RF output power (adaptive frequency)	0.53dBm (2402MHz)
	Communication distance	Approx. 5m

Design and specification are subject to change without advanced notice.


## Setting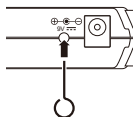

Press the button

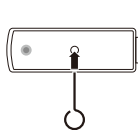Press the button
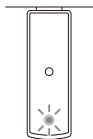
## 1. Additional pairing

Set up the **device adapter** on the body composition analyzer and press the button. Next, set up the **PC adapter** on your computer and press the button. The lamps blink quickly in green while in pairing mode.

2. When pairing is completed, the adapters are connected and the lamps remain lit in green.

**Note**

The **device adapter** can be paired with multiple **PC adapters**. Once paired, the pairing is retained after power is turned off.

The **PC adapter** can only be paired with a single **device adapter**. Pairing must be performed each time a different **device adapter** is used.

**Note**

The adapter lamps blink red if pairing is not completed.

**⚠ Caution**

Prohibited

- Do not place the device in hot or humid locations, locations with extreme variations in temperature, direct sunlight, dirty or dusty locations or locations where it may get wet.
- Make sure to insert the power plug and all other connectors completely.
- Do not unplug the power plug or any other connector by pulling on the cord.
- Because this device is electronic, do not subject it to static electricity.
- Unplug the AC adapter and refrain from using this device around any electronic equipment that handles high-precision control or weak signals.  
Failure to heed this caution may cause such devices to malfunction or be otherwise affected.
- Do not use this device near any device that generates electromagnetic waves.  
Doing so may impair proper operation or lower the communication range.
- Do not use benzine, thinner or any similar product for cleaning. Instead, soak a cloth in water or neutral household cleaner, wring out excess liquid, then wipe dirt away. Finish by wiping with a dry cloth.
- Do not use extension cables or similar cables to connect this device to TANITA equipment or PC.
- Dispose of this device as indicated by local law.

**Troubleshooting**

Problem	Solution
No LED lights.	Power supply error. Make sure the RS-232C and USB are fully plugged in. Make sure that the TANITA equipment power supply is connected and the power sharing setting is set to on, then connect again. See the TANITA website for how to set power supply on the TANITA equipment: <a href="http://www.tanita.eu">www.tanita.eu</a> Incorrect AC adapter. Use the designated AC adapter. Check the computer power supply.
LED light is blinking in red.	Pairing has failed. Reconnect this device or press the switch again. (See "Setting")
LED light turns red.	<i>Bluetooth</i> <sup>®</sup> is disconnected. Additionally, make sure there is nothing nearby that could interfere with the <i>Bluetooth</i> <sup>®</sup> signal. Wait for a short while, making sure the area remains free of interferences.
LED light blinks alternating red and blue, and measurement results are not displayed in the app.	The signal may have timed out. Check the connection between this device and the TANITA equipment.

Design and specification are subject to change without advanced notice.

Hereby, TANITA declares that the radio equipment type [OP-102] is in compliance with EU Directives and UK Regulations for Radio Equipment and RoHS.

The full text of the EU and UK declaration of conformity is

available at the following internet address:

<https://www.tanita.eu/help-guides/products-manuals/>

**Disposal**

This is an electronic device.

Please dispose of it as an electronic device, not as general household waste. Please follow the regulations in your local region when disposing of this device.

- The product design and specifications may be changed at any time without prior notice.
- The *Bluetooth*<sup>®</sup> word mark and logos are registered trademarks owned by the *Bluetooth* SIG, Inc. and any use of such marks by TANITA Corporation is under license. Other trademarks and trade names are those of their respective owners.

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### **TANITA (Shanghai) Trading Co., Ltd.**
### **TANITA Corporation of America, Inc.**
### **UK Representative TANITA Europe B.V.**