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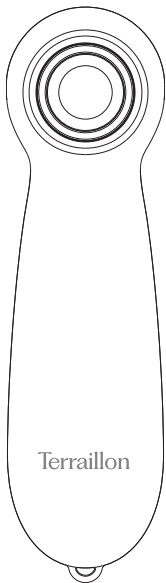
Dear Customer,

Thank you for purchasing INSECT CALM. Please read the instructions before using the device as this manual provides additional information including safety precautions and warnings.

If you have any difficulties or need more information, you can find our help centre by scanning the QR code opposite with your smartphone. If you cannot find the answer to your question, you can also contact our After Sales Service: support.uk@terraillon.com

INTRODUCTION

- In order to use the device safely, read the complete manual carefully before using the device for the first time.
- Keep this instruction manual in a convenient place, or store with the device for future reference.
- The device is a medical device designed to be used in the home healthcare environment.
- It is suitable for use by all who can control the device and understand the instructions.
- Patient groups: 3 years old and older.
- Intended users: Lay users, for children under the age 12 should be used under adult supervision.

OPERATING PRINCIPLE

- Insect calm can relieve the insect bite/sting skin by concentrating heat at about 51°C on the affected area. The mechanism of action of applying concentrated heat (local hyperthermia) is based upon the brief, concentrated effect of heat on a very small area of skin.
- Even a locally limited application of heat may be enough to trigger a pain and itch-relieving reaction of the body. This means that once the device heats to the right temperature and you use it on the area of skin affected by the sting or bite, that heat transfer quickly reduces skin itching, pain and swelling.

INDICATIONS FOR USE

Insect calm is intended for symptomatic treatment of itching, pain and swelling caused by insect stings and bites.

STANDARD PARTS

The package contains the following components:

No.	DESCRIPTION	QUANTITY
A	Insect calm	x 1
B	User Manual	x 1
C	Batteries, type AAA (LR03 1.5 V)	x 2

IMPORTANT SAFETY PRECAUTIONS AND WARNINGS

It is important that you read all the contraindications, warning and precautions included in this manual because they are intended to keep you safe, prevent injury and avoid a situation that could result in damage to the device.

SAFETY SYMBOLS USED IN THIS MANUAL

WARNING

Indicates a potentially hazardous situation which, if not avoided, could result in serious injury and equipment damage.

CAUTION

Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user, or damage to the device or other property.

CONTRAINDICATIONS

- Do not use if you suffer from diabetes, The pain may have diminished, and diabetics may burn themselves.
- Not for use on children under the age of 3.
- Do not use on inflamed skin, reddened skin, irritated skin, rashes, burns, frostbite, bruises, swellings, open or healing wounds, unhealed or fresh post-operative scars.

- Do not use, if suffering with fever.
- Do not use the device on sensitive areas of skin.
- Do not use, if you have lotions, creams, and gels on your skin at the time of insect bite.
- Do not use, if your perception of pain is reduced (i.e. metabolic disorders).
- Do not use if you suffer from persistently inflamed skin due to long term health application to the same skin area (i.e. where the insect bite occurred).

GENERAL WARNING

- The patient may be the proper operator of the Insect calm.
- Not for use on children under the age 12 without adult supervision.
- Not for use by disabled people and people with reduced sensitivity to heat, such as people with illness related skin alterations and scars in application area.
- Do not use if you have thromboembolism or recurrent malignant growth or unexplained chronic pain in any area of the body."
- Do not use near the eyes, eyelids, mouth, or mucus membranes."
- Ensure that you observe general rules of hygiene, when using the Insect calm.
- Seek medical advice if bite symptoms persist or worsen despite using the Insect calm.
- Seek medical advice immediately, if the bite is a tick bite as it may result in a transmission of pathogens (i.e. tick-borne encephalitis (TBE) or borreliosis.

- Seek medical advice if the insect bite related symptoms, such as itching, redness, and skin swelling, dizziness, nausea, shortness of breath, formation of wheals, or drop or/increase in blood pressure occurs.
- If in doubt as to your suitability for using Insect calm, consult a medical practitioner (i.e. a nurse or a doctor).
- Make sure children do not treat the device as a toy.
- Keep equipment out of reach of children and pets.
- To avoid the risk of choking, keep the packaging out of the reach of children.
- Observe general hygiene practices when using the equipment.
- This device is not a substitute for medical consultation and treatment. So consult your doctor if you have any pain or illness.
- If the device does not work properly, if you feel uncomfortable or painful, please stop using it immediately.
- Do not service and maintain while the device is in use.
- Do not modify the device without authorization of the manufacturer. This could cause improper functioning. You can replace battery and clean the device, other maintenance is not allowed.
- If you have an allergic reaction to materials, please stop treatment immediately and consult your physician.
- Please keep the battery compartment closed during operation and avoid touching the battery and the body part at the same time.

MATTERS NEEDING ATTENTION

- Do not touch the device with wet hands. Do not inject water into the device. The equipment can only be operated in a completely dry state.
- Do not use detergent or solvent to clean or maintain the device. The injected liquid can damage electronics and batteries and may cause device failure.
- The device can only be used by one person. Multiple users are not recommended.
- Protect the device from direct sunlight and store it dry.
- Protect equipment from shock, moisture, dust, chemicals, severe humidity fluctuations, and near heat sources (furnaces, radiators).
- Make sure no water enters the equipment. If this happens, remove the battery and use it only when the device is completely dry.
- Do not place heavy objects on the storage device.
- Make sure there is no visible damage before use. If in doubt, do not use, please contact your dealer or customer service address.
- Equipment must be maintained in expected environmental conditions.
- Do not leave the device unattended during use. If something goes wrong, stop using it immediately.

BATTERY PRECAUTIONS

Warning

- If battery leakage comes into contact with skin or eyes, wash the affected area with water and seek medical advice.
- Please keep the battery away from children. There is a risk of suffocation if swallowed.
- Keep away from the source of fire, which could cause an explosion.
- When the battery runs out and leaks, wear protective gloves and clean the battery compartment with a dry cloth.
- Do not remove, open or crush the battery.

CAUTION

- Note the polarity plus (+) and minus (-) signs.
- Protect the battery from heat.
- Batteries are not allowed to be charged or short-circuited.
- If the device is not used for a long time, take the battery out of the battery compartment.
- Use only the same battery.
- Be sure to replace all batteries at the same time.

SIDE EFFECTS

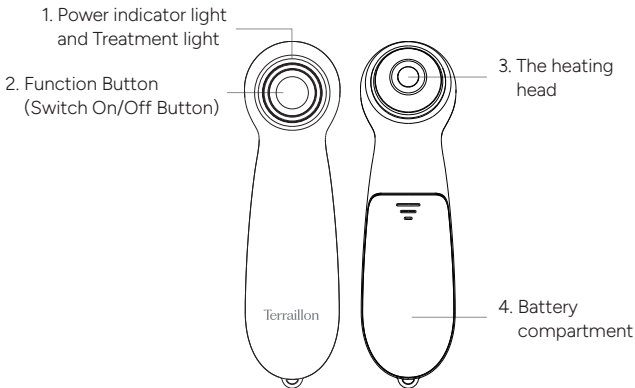
Depending on sensitivity, the treatment temperature may cause some discomfort. In sensitive individuals, a temporary reddening of the skin may occur after use. In rare cases, users with sensitive skin types may experience skin irritation.

RESIDUAL RISK

The heated ceramic surface may reach a maximum temperature of 127°F (53°C). You may get skin burn if used on the same skin area more than once within a short period of time.


KNOW YOUR DEVICE

Front and rear panel

INSERTING BATTERIES

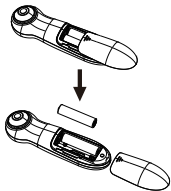1. Remove the battery cover on the back of the device as shown by the graphic to the right.
2. Insert 2xAAA batteries. Make sure the positive (+) and negative (-) signs correspond with the markings in the device when inserting batteries.

3. Reinstall the battery cover as shown by the graphic to the right.

Notes:

- Please use 2xAAA batteries in this unit.
- Remove the batteries if the device is not in use for long periods of time.
- Do not mix old and new batteries or different types of batteries.
- Warning: If batteries leak and come into contact with the skin or eyes, wash immediately with large amounts of water.
- Batteries must be handled by an adult. Keep batteries out of the reach of children.
- Remove exhausted batteries from the unit.
- Dispose the used batteries safely according to local regulations.

EASY STEPS TO GET STARTED WITH INSECT CALM

1. Prior to applying Insect calm, ensure that the sting area is clean and that all remains of the sting have been carefully removed. Ensure that the device is cleaned before and after each use.

For best outcomes, use Insect calm immediately after the sting or bite occur. However, even at the later stage after the sting, Insect calm can still be helpful for symptomatic relief and feeling of greater comfort.

2. Ensure that the batteries are fitted in the battery compartment and the compartment lid is closed.
3. Long press the button(2) to power up – you will see the indicator LED light up green(1).

Caution:

- The time required for the device to warm from the minimum storage temperature (-10°C) between use until the device is ready for use at ambient temperature 20°C: about 2 hours.
 - The time required for the device to cool from the maximum storage temperature (55°C) between use until the device is ready for use at ambient temperature (20 °C) : about 2 hours.
4. Select an appropriate programme:
 - for sensitive skin or for first use – select 3 seconds programme by pressing the function button once (1 x)
 - for normal skin and subsequent applications – select 6 seconds programme by pressing the function button twice (2 x)

Caution:

After selecting the corresponding programme, the device starts to warm up, which takes a few seconds (it determined by the current temperature of the Heating head(3)). When Heating head(3) reaches the target temperature and then start the treatment according to the selected programme. When using Insect calm for the first time, you may opt to use it on the 3 seconds programme to enable you to test your tolerance of the heat treatment.

5. At the end of the application, the device sounds an acoustic signal and the illuminated ring goes out. Remove the hot plate from the skin as soon as the acoustic signal sounds. Heating of the hot plate automatically stops.
6. If you do not intend to use the device, Long press the button to the power off.

Caution:

Alternatively, you can re-use the device on another sting. However, please note that the maximum number of 5 applications per hour on the same treatment area must not be exceeded.

7. Ensure you clean the device after use.

Caution:

- Ensure that you follow cleaning instructions outlined in user manual. Insect calm is a multiple use device.
- In the interest of hygiene and prevent potential cross contamination, the device must be cleaned before and after each use. Consult user manual for details and helpful advice.

- Ensure that Insect calm is switched off, cool and clean prior to placing in storage. If you intend to store it unused for a long time, it is best to remove batteries from the compartment for the duration of storage.

OTHER IMPORTANT FUNCTIONS IN INSECT CALM

Automatic shutdown

If no operation is performed for more than one minute, the device automatically shuts down.

Low electricity prompt

If the voltage of the point battery is too low, three prompt tones will sound and the power indicator will blink at 1Hz, Automatic shut down after 3 seconds.

Abnormal temperature protection

- The device cannot be used when the temperature of the device is lower than 5°C or higher than 53°C (or the temperature sensor is damaged).
- When these abnormal conditions occur, the treatment light will flash in red light to remind. When the device temperature is ready for use, restart the device for treatment.

TECHNICAL SPECIFICATIONS

Product Name	Insect Calm / BITE GONE
Model	CH3000
Type	SE
Power supply:	2 x 1.5 V AAA LR03 batteries
Maximum temperature:	51°C ±2°C
Functions	2 x settings (3 seconds / 6 seconds)
Expected operating duration	circa 300 applications
Operating conditions	10°C to 40°C, RH 30% - 75%; 700hPa-1060hPa
Storage conditions	-10°C to 55°C, RH 10% - 90% 700hPa-1060hPa
Dimensions	119 x 32.5 x 30.1 mm
Weight	about 50g (with batteries)
IP Classification	IP22
Applied part	Heating head
Classification of ME Equipment	Internally powered/ Continuous
Service life of the device	3 years
Service life of the battery	New batteries will last for approx.300 times (3 seconds program)
Software version	V1

CLEANING AND STORAGE

Cleaning the device

- Please observe hygiene guidelines at all times.
- It is recommended to clean the device before and after each use.
- Please shut down the device and remove the batteries before cleaning.
- The device temperature must be restored to room temperature during cleaning.
- Use a soft towel (a small amount of alcohol or other medical cleanser is recommended) to wipe the heater and the casing.
- Do not let detergent into the machine, Irreversible damage may be caused to the device.
- Do not immerse the device in detergent, Irreversible damage may be caused to the device.
- Do not use chemicals (such as benzene, thinner).
- Do not clean in washing machine or other cleaning equipment.
- Do not use hard brushes to clean the equipment, which may cause scratches.
- If the device is used between different skin areas, clean the device with alcohol or other medical cleanser to prevent the spread of bacteria from one skin area to another.

Storing the device

- Place the unit, and manual back into retail box. Store the box in a cool, dry place, 14°F~131°F (-10°C~55°C); 10%~90% relative humidity. Failing to do so may affect the performance of the device.
- Do not store in places that can be easily reached by children. Store the device in the dry, clean place. It is detrimental to leave the product exposed to any chemical solvent, lint, dust, direct sunshine or high temperature.
- When not in use for a long period, remove the batteries before storage to avoid liquid discharge from batteries.

DISPOSAL

- Used fully discharged batteries must be disposed of in a specially labeled collection container, at a toxic waste collection point or through an electrical retailer. You are under legal obligation to dispose of batteries correctly.
- Please dispose of the device in accordance with local regulations.

WARRANTY

- This product is guaranteed against defects in materials or manufacturing. During the guarantee period, any such defects will be replaced free of charge (proof of purchase must be shown in the case of a complaint under guarantee). The guarantee does not cover damage resulting from improper use or negligence. In the event of a claim, first contact the shop where you purchased the product.
- European Medical Device Regulation requires that any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority in your country.

TROUBLESHOOTING

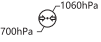
If the unit does not operate after taking these measures, contact the nearest dealer.

PROBLEM	POSSIBLE CAUSES	POSSIBLE SOLUTION
The unit cannot power on	Are the batteries exhausted?	Replace the batteries
Equipment not hot enough	Battery low voltage	Replace the batteries
The power indicator blinks three times, and then the machine shuts down	Battery low voltage	Replace the batteries
Although the battery has been replaced, it still won't turn on	The wiring may be defective	Please contact customer service
Although the device temperature is ready for use, treatment light still flash in red light	The temperature sensor is damaged	Contact the manufacturer or dealer

GLOSSARY OF SYMBOLS

	<p>Electrical devices are recyclable material and should not be disposed of with household waste after their useful life. Help us protect the environment and save resources and by taking this device to the appropriate collection point. Please contact the organization which is responsible for waste disposal in your area if you have any questions.</p>
	<p>Caution</p>
	<p>Type BF Applied Part</p>
	<p>Please refer to instruction manual</p>
<p>IP22</p>	<p>The first number 2: Protected against solid foreign objects of 12,5mm \varnothing and greater. The second number: Protected against vertically falling water drops when enclosure tilted up to 15°. Vertically falling drops shall have no harmful effects when the enclosure is tilted at any angle up to 15° on either side of the vertical.</p>
	<p>Serial number</p>
	<p>The name and the address of the manufacturer</p>
	<p>Transportation and storage temperature from -10°C to 55°C</p>

EN

	Transportation and storage humidity limits from 10% to 90%
	Transportation and storage atmospheric pressure limits from 700hPa to 1060hPa
	Date of Manufacture
	The name and the address of the Authorized EC-representative in Europe
	The name and the address of the authorized representative in Swiss
	The name and the address of the authorized importer in Switzerland
	Medical Device
	Unique device identifier
	CE mark

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IMPORTANT INFORMATION REGARDING ELECTROMAGNETIC COMPATIBILITY (EMC)

- This device should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, this device should be observed to verify normal operation in the configuration in which it will be used.
- Use of accessories, detachable parts and materials other than those specified or provided by the manufacturer of this device could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- When the operating environment is relatively dry, strong electromagnetic interference usually occurs. At this time, the device may be affected as follows:
 - the device stops output;
 - the device turns off;
 - the device restarts;

The above phenomenon does not affect the basic safety and essential performance of the device, and the user can use it according to the instruction. If you want to avoid the above phenomenon, please use it according to the environment specified in the manual.

Table 1

Declaration - electromagnetic emission		
The device is intended for use in the electromagnetic environment specified below. The customer or the user of device should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Not applicable	

Table 2

Declaration - electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 0.5kV, ± 1 kV line(s) to lines ± 0.5kV, ± 1 kV, ± 2 kV line(s) to earth	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: at 0° 0 % UT; 250/300 cycles	Not applicable	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: UT is the a.c. mains voltage prior to application of the test level.			

Table 3

Declaration - electromagnetic immunity			
The thermometer are intended for use in the electromagnetic environment specified below. The customers or the users of these thermometer should assure that it is used in such environment.			
Immunity test	IEC 60601 Test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3V 0.15 MHz to 80MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz	Not applicable	<p>Portable and mobile RF communications equipment should be used no closer to any part of device, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = 1.2 \sqrt{P} \text{ 150 KHz to 80 MHz}$ $d = 1.2 \sqrt{P} \text{ 80 MHz to 800 MHz}$ $d = 2.3 \sqrt{P} \text{ 80 MHz to 2.7 GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p>
Radiated RF IEC 61000-4-3	10V/m 80 MHz to 2.7 GHz	10 V/m	<p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.</p> <p>^b Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

Note1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note2: These guidelines may not apply in all situations.

Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

^a Field strengths from fixed RF transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which device is used exceeds the applicable RF compliance level above, device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating device.

^b Over the frequency range 0.15 MHz to 80 MHz, field strengths should be less than 3 V/m.

Table 4

Recommended separation distances between portable and mobile RF communications equipment and device

The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and device, as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	80 MHz to 2.7 GHz $d = 2.3 \sqrt{P}$
0.01	0,12	0,04	0,07
0.1	0,38	0,11	0,22
1	1,2	0,35	0,7
10	3,8	1,1	2,2
100	12	3,5	7

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Terrailon SAS France & Headquarters

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Terrailon Asia Pacific Ltd

4/F, Eastern Centre
1065 King's Road
Quarry Bay – Hong Kong
Tel: +852 (0)2960 7200
customerservice@terraillon-asia.com

Terrailon Corp USA

contact@terraillon.fr

Terrailon UK

support.uk@terraillon.com

Terrailon®

www.terraillon.com